

# Urinalysis Reagent Strips

## Simple and Accurate


- Analytical sensitivity better than or comparable to market leaders
- High quality color chart ensures accurate visual reading

## Flexible

- Compatible for visual and analyzer reading
- Over 35 different combinations available

## Multiple Packaging Options and Long Shelf Life


- Canister Packaging**
  - Available in 25, 50 and 100 strips per canister
  - 2 year shelf life for unopened canisters which 150 strips per kit without MA/CRE Combo offers cost savings and convenience for high volume testing
  - 3 month shelf life for strips in opened canisters
- Pouch Packaging**
  - Individually packaged strips available in kit of 3 or 6 strips for visual reading only (includes 1 color chart)
  - Unique packaging maintains 2 year shelf life for all strips in the kit compared to 3 months for remaining strips in an opened canister

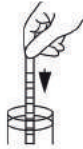

**Step 1: Immerse strip into urine**



**Step 2: Remove excess urine**

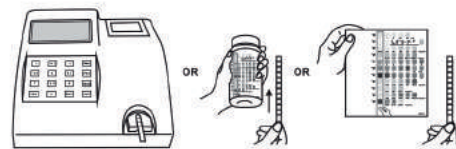

**Step 3: Obtain results by analyzer or visual reading**

| No. | Catalog No. | No. of Parameters | Type of Strip <sup>§</sup> |                  | Reading Availability |      |            |      | Parameters |     |     |     |    |     |    |     |     |     |     |     |     |    |
|-----|-------------|-------------------|----------------------------|------------------|----------------------|------|------------|------|------------|-----|-----|-----|----|-----|----|-----|-----|-----|-----|-----|-----|----|
|     |             |                   | Visual Reading             | Analyzer Reading | Visual               | U120 | U120 Ultra | U500 | ASC        | GLU | BIL | KET | SG | BLO | PH | PRO | URO | NIT | LEU | ALB | CRE | CA |
| 1   | U031-141    | 14                | 14C✓                       |                  | Yes                  | Yes  | Yes        | Yes  | *          | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 2   | U031-131    | 13                | 13CE✓                      |                  | Yes                  | Yes  | Yes        | Yes  | *          | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 3   | U031-111    | 11                | 11A✓†                      |                  | Yes                  | Yes  | Yes        | Yes  | *          | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 4   | U031-101    | 10                | 10U✓x                      |                  | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 5   | U031-191    | 9                 | 9U✓x                       |                  | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 6   | U031-081    | 8                 | 8U✓x                       |                  | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 7   |             |                   | 8N✓x                       |                  | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 8   |             |                   | 8S✓x                       |                  | Yes                  | Yes  | No         | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 9   |             |                   | 8K✓x                       |                  | Yes                  | Yes  | No         | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 10  | U031-071    | 7                 | 7N✓x                       |                  | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 11  | U031-061    | 6                 | 6N✓x                       | 6NE✓x            | Yes                  | Yes  | No         | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 12  |             |                   | 6U✓x                       | 6UE✓x            | Yes                  | Yes  | No         | Yes  |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 13  | U031-051    | 5                 | 5B✓x                       | 5BE✓x            | Yes                  | Yes  | No         | No   |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 14  |             |                   | 5N✓x                       | 5NE✓x            | Yes                  | Yes  | Yes        | No   |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 15  |             |                   | 5S✓x                       | 5SE✓x            | Yes                  | Yes  | No         | No   |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 16  |             |                   | 5U✓x                       | 5UE✓x            | Yes                  | Yes  | No         | No   |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 17  | U031-041    | 4                 | 4P✓x                       | 4PE✓x            | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 18  |             |                   | 4S✓x                       | 4SE✓x            | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 19  |             |                   | 4B✓x                       | 4BE✓x            | Yes                  | Yes  | No         | No   |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 20  |             |                   | 4K✓x                       | 4KE✓x            | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 21  |             |                   | 4G✓x                       | 4GE✓x            | Yes                  | Yes  | No         | No   |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 22  |             |                   | 4N✓x                       | 4NE✓x            | Yes                  | Yes  | No         | Yes  |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 23  | U031-031    | 3                 | 3P✓x                       | 3PE✓x            | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 24  |             |                   | 3K✓x                       | 3KE✓x            | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 25  |             |                   | 3G✓x                       | 3GE✓x            | Yes                  | Yes  | No         | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 26  |             |                   | 3N✓x                       | 3NE✓x            | Yes                  | Yes  | No         | Yes  |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 27  | U031-021    | 2                 | 2G✓x                       | 2GE✓x            | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 28  |             |                   | 2K✓x                       | 2KE✓x            | Yes                  | Yes  | Yes        | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 29  |             |                   | 2N✓x                       | 2NE✓x            | Yes                  | Yes  | Yes        | Yes  |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 30  |             |                   | 2B✓x                       | 2BE✓x            | Yes                  | Yes  | No         | Yes  |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 31  |             |                   | 2U✓x                       | 2UE✓x            | Yes                  | Yes  | No         | Yes  |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 32  |             |                   | 2S✓x                       | 2SE✓x            | Yes                  | Yes  | No         | Yes  |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 33  |             |                   | 2C✓                        | 2CE✓             | Yes                  | Yes  | Yes        | Yes  |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 34  | U031-011    | 1                 | 1B✓x                       | 1BE✓x            | Yes                  | Yes  | No         | No   |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 35  |             |                   | 1P✓x                       | 1PE✓x            | Yes                  | Yes  | No         | No   |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 36  |             |                   | 1G✓x                       | 1GE✓x            | Yes                  | Yes  | Yes        | No   |            | *   | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 37  |             |                   | 1K✓x                       | 1KE✓x            | Yes                  | Yes  | No         | No   |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |
| 38  |             |                   | 1R✓x                       | 1RE✓x            | Yes                  | Yes  | No         | No   |            |     | *   | *   | *  | *   | *  | *   | *   | *   | *   | *   | *   | *  |

### §Type of Strip:

Visual Strip Size: 1-6 Parameters: 80 mm x 5 mm; 7-14 Parameters: 108 mm x 5 mm

U120/U500 Strip Size: 1-14 Parameters: 108 mm x 5 mm

"E" means extended strip length for 1-6 Parameters and exclusive strip length for 13 Parameter

Default Type of Strip (U120/U500): 11A, 10U, 9U and 8N

Standard Black Canisters : Available for 25, 50 and 100 strips; 150 strips per kit without MA/CRE Combo

Pouch: Single-strip pouch available in kit of 3 or 6 for visual reading only

✓ CE Marked for sale in the European Community

† FDA 510(k) Cleared

x FDA 510(k) Cleared and CLIA Waived



# U120 Ultra Urine Analyzer



## Easy to Operate

- Large color touchscreen LCD for simple menu navigation
- Work List and Help Menu available for specimen review and troubleshooting
- Powered by AC adaptor or 6 AA batteries for easy portability
- Up to 2,000 patient memory and 800 Operator ID storage
- Ability to select Time Logout between 1-99 with minutes or hours option

## Accurate and Efficient

- Advanced CMOS Image Sensor ensures accurate readings
- Can read strips with up to 14 parameters, including Microalbumin, Creatinine and Calcium
- Option to edit test number sequence, or skip then return to specific test numbers
- Ability to edit abnormal results

## Simple Data Transfer

- Immediate transmission of LIS data using Bluetooth, LAN or WLAN
- Ability to update software with SD card or USB flash drive

## Unique Lockout Functions

- Strip Lockout
  - Pre-set option to prevent using strips of another brand, with a barcode reader or by manual entry
- User Lockout
  - Option to eliminate unapproved users with up to 800 operators
- QC Lockout
  - Prevents testing without passing QC
  - If QC tests fail, analyzer will switch to STAT mode and list "E" at the end of each test number

## Specifications

| Feature                           | Specifications                                                                                                                                                                                                                                                                                                                                                     |
|-----------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Analyzer Type                     | Manual                                                                                                                                                                                                                                                                                                                                                             |
| Methodology                       | Reflectance Photometry                                                                                                                                                                                                                                                                                                                                             |
| Detection                         | CMOS Image Sensor                                                                                                                                                                                                                                                                                                                                                  |
| Throughput                        | Single Test Option: 55 tests/hour; Continuous Test Option: 120 tests/hour                                                                                                                                                                                                                                                                                          |
| Test Modes                        | Quick Test Mode, Full Test Mode and Customized Test Mode                                                                                                                                                                                                                                                                                                           |
| Test Category                     | Routine, STAT and QC                                                                                                                                                                                                                                                                                                                                               |
| Lockout Functions                 | Strip Lockout: Available Upon Request; User/QC Lockout: Included with option to turn ON/OFF                                                                                                                                                                                                                                                                        |
| Memory                            | Last 2,000 Records                                                                                                                                                                                                                                                                                                                                                 |
| Strip Incubation Time             | 1 Minute                                                                                                                                                                                                                                                                                                                                                           |
| Wavelength                        | 390 nm - 770 nm                                                                                                                                                                                                                                                                                                                                                    |
| Strips Available                  | 1-14 parameters (108 mm x 5 mm); see URS Parameters                                                                                                                                                                                                                                                                                                                |
| Parameter Order                   | Can select the order of parameters for display and print out                                                                                                                                                                                                                                                                                                       |
| Total Combinations Per Analyzer   | Over 15 Combinations                                                                                                                                                                                                                                                                                                                                               |
| Analyzer Ports                    | RS232C Port for Barcode Reader or Data Transfer and External Printer<br>USB Ports for Keyboard or Data Transfer                                                                                                                                                                                                                                                    |
| Data Entry Capabilities           | Operator ID, Patient ID/Name - Manual Entry and Barcode Reader (Up to 20 characters)<br>Urine Color and Clarity, Strip Lot Number, and Expiration Date - Manual Entry                                                                                                                                                                                              |
| Connection Capabilities           | Internal Thermal Printer (included)<br>Bluetooth (included)<br>Bluetooth Adaptor (optional)<br>RS232C Barcode Reader (optional)<br>USB or RS232C Data Transfer Cable (optional)<br>SD Card or USB flash drive for Software Update (optional)<br>Ethernet via USB to RJ45 Adaptor (optional)<br>Keyboard (not included)<br>Optional External Printer (not included) |
| Major Readable Barcodes           | Code 39      EAN 8      French Pharmacode      Matrix 25      RSS<br>Code 93      EAN 13      Industrial 25      MSI      Telepen<br>Code 128      EAN 128      Interleave 25      Plessey      UPCA<br>Codabar (NW-7)      Italy Pharmacode      UPCE                                                                                                             |
| Screen Type                       | Large color touch screen LCD (12 cm x 9 cm)                                                                                                                                                                                                                                                                                                                        |
| LIS Interface                     | Formatted and compatible with HL-7 compliant, ACON standard interface, S interface, D interface, U interface and R interface for downloading of LIS data                                                                                                                                                                                                           |
| Calibration                       | Automatic                                                                                                                                                                                                                                                                                                                                                          |
| Available Languages on the Screen | More than 10 languages available, including English                                                                                                                                                                                                                                                                                                                |
| Analyzer Operating Conditions     | 0-40°C (32-104°F); 5%-85% RH                                                                                                                                                                                                                                                                                                                                       |
| System Operating Conditions       | 15-30°C (59-86°F); 20%-80% RH                                                                                                                                                                                                                                                                                                                                      |
| Storage Conditions                | -5-50°C (23-122°F); ≤90% RH                                                                                                                                                                                                                                                                                                                                        |
| Power Source                      | 100- 240 VAC, 45-65 Hz; 6 AA Alkaline Batteries                                                                                                                                                                                                                                                                                                                    |
| Line Leakage Current              | 0.5 mA                                                                                                                                                                                                                                                                                                                                                             |
| Dimensions (L x W x H)            | 26 cm x 15 cm x 18 cm (10" x 6" x 7")                                                                                                                                                                                                                                                                                                                              |
| Display Dimensions (L x W)        | 12 cm x 9 cm (5" x 4")                                                                                                                                                                                                                                                                                                                                             |
| Weight                            | 1.7 kg (3.7 lb) without batteries or power supply                                                                                                                                                                                                                                                                                                                  |

## Ordering Information

| Product Name                                  | Catalog No. | Components                                                                                                      |                                                                                                   | Kit Box Dimensions (L x W x H) & Weight                                                                                                | Carton Dimensions (L x W x H) & Weight                                            | Number of Kits/Carton |
|-----------------------------------------------|-------------|-----------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|-----------------------|
| U120 Ultra Urine Analyzer                     | U114-101 ✓  | 1 Urine Analyzer<br>2 Test Tables<br>2 Test Table Inserts<br>2 Printer Paper Rolls                              | 1 Power Cord and Supply Adapter<br>1 Brush<br>1 Quick Start Guide<br>1 Instruction Manual         | 40 cm x 39 cm x 36 cm; 4 kg                                                                                                            | 16" x 15" x 14"; 141 oz                                                           | 1                     |
| U120 Ultra Urine Analyzer with Barcode Reader | U114-111 ✓  | 1 Urine Analyzer<br>2 Test Tables<br>2 Test Table Inserts<br>2 Printer Paper Rolls<br>1 Barcode Reader (RS232C) | 1 Power Cord and Supply Adapter<br>1 Brush<br>1 Quick Start Guide<br>1 Instruction Manual         | 40 cm x 39 cm x 36 cm; 4 kg                                                                                                            |                                                                                   |                       |
| Barcode Reader                                | U124-111 ✓  | 1 Barcode Reader (RS232C)                                                                                       |                                                                                                   | 23.6 cm x 10.8 cm x 7.8 cm; 0.36 kg<br>9.3" x 4.3" x 3.1"; 17.0 oz                                                                     | 63.0 cm x 37.0 cm x 30.0 cm; 19.4 kg<br>24.8" x 14.6" x 11.8"; 684.3 oz           | 22                    |
| Printer Paper Rolls                           | U121-101    | 4 Printer Paper Rolls                                                                                           | Thermal Paper (0.06 m x 20 m): 200 results/roll<br>Sticker Paper (0.06 m x 9 m): 100 results/roll | 12.0 cm x 12.0 cm x 6.5 cm; 0.36 kg<br>4.7" x 4.7" x 2.6"; 12.7oz<br>12.0 cm x 12.0 cm x 6.5 cm; 0.4 kg<br>4.7" x 4.7" x 2.6"; 14.1 oz | 63.0 cm x 37.0 cm x 30.0 cm; 21.4 kg<br>24.8" x 14.6" x 11.8"; 684.3 oz; 754.9 oz | 50                    |
| U120 Ultra Data Transfer Kit                  | U124-131 ✓  | 1 Data Transfer Cable (RS232C)                                                                                  | 1 Package Insert                                                                                  | 16.0 cm x 13.0 cm x 3.5 cm; 0.147 kg<br>6.3" x 5.1" x 1.4"; 5.2 oz                                                                     | 25.0 cm x 21.0 cm x 15.0 cm; 1.36 kg<br>9.8" x 8.3" x 5.9"; 48.0 oz               | 8                     |

# Urine Controls

## Reliable

- Use with *Mission®* and *Mission® Expert* Urinalysis Reagent Strips and Urine Analyzers for optimum quality control
- Validate urinalysis results and prevent procedure errors

## Quick and Convenient Testing

- Ensures accurate results for all parameters
- Obtain quick results in any setting
- Competitively priced



## Two Types of Urine Controls Available

### Liquid Urine Control

- Ready to use without dissolving in distilled water
- 24 months shelf life for unopened controls at 2-8°C
- Two Packaging Options
  - Dropper Tip Bottles
    - Dropper tip bottles provide efficient use of the control solution
    - Easily drop the control solution onto each reagent pad using the dropper tip bottle
    - Controls can be used up to 40 times within 30 days at room temperature
  - Driptubes
    - Driptube packaging allows for quick testing similar to using a urine specimen
    - Simply dip the strip into the control solution and read results
    - Controls can be used up to 20 times within 30 days at room temperature

### Dry Strip Urine Control

- Portable for use anywhere with no refrigeration required
- Dissolve the dry strip urine control in distilled water, dip urine strip in the control solution, then compare to color chart
- Each control solution can be used for up to 12 tests at 2-30°C within 8 hours for all parameters
- 24 months shelf life at 2-30°C for unopened controls

## Specifications

| Features                    |         | Specifications                                                                              |                                                          |                                                                                 |
|-----------------------------|---------|---------------------------------------------------------------------------------------------|----------------------------------------------------------|---------------------------------------------------------------------------------|
| Product Name                |         | Liquid Urine Control                                                                        | Liquid Driptube Urine Control                            | Dry Strip Urine Control                                                         |
| Test Parameters             |         | LEU, NIT, URO, PRO, pH, BLO, SG, KET, BIL, GLU, ASC, ALB, CRE, CA (13)                      |                                                          |                                                                                 |
| Solution Detection Levels   | Level 1 | Negative: LEU, NIT, URO, PRO, pH, BLO, SG, KET, BIL, GLU, ASC, ALB, CRE, CA                 |                                                          |                                                                                 |
|                             | Level 2 | Positive: LEU, NIT, URO, PRO, pH, BLO, SG, KET, BIL, GLU, ALB CRE, CA and Negative ASC      |                                                          |                                                                                 |
| Compatible Urine Strips     |         | <i>Mission®</i> Urinalysis Reagent Strips, <i>Mission® Expert</i> Urinalysis Reagent Strips |                                                          |                                                                                 |
| Reading Time/Stability      |         | Refer to insert                                                                             | Refer to insert                                          | Refer to insert                                                                 |
| Storage Temperature         |         | 2-8°C                                                                                       | 2-8°C                                                    | 2-30°C                                                                          |
| Unopened Control Shelf Life |         | 24 months                                                                                   | 24 months                                                | 24 months                                                                       |
| Opened Control Stability    |         | 30 days at 15-30°C or until the expiration date at 2-8°C                                    | 30 days at 15-30°C or until the expiration date at 2-8°C | 2-30°C: 3 months for Dry Strip; 8 hours for Control Solution for all parameters |
| Maximum Tests per Unit      |         | 20 or 40 tests/bottle                                                                       | 20 tests/driptube                                        | 12 tests/control solution of 1 dry strip                                        |

## Ordering Information

| Product Name                     | Catalog No. | Components                                                       | Kit Box Dimensions (LxWxH) & Weight | Carton Dimensions (LxWxH) & Weight | # Kits/Carton |
|----------------------------------|-------------|------------------------------------------------------------------|-------------------------------------|------------------------------------|---------------|
| Liquid Urine Control ✓X          | U021-011    | Level 1: 3 x 10 mL/bottle; Level 2: 3 x 10 mL/bottle             | 85 mm x 55 mm x 60 mm; 107 g        | 400 mm x 270 mm x 345 mm; 5.2 kg   | 198           |
|                                  |             | Level 1: 3 x 5 mL/bottle; Level 2: 3 x 5 mL/bottle               | 85 mm x 55 mm x 60 mm; 75 g         | 400 mm x 270 mm x 345 mm; 4.2 kg   | 198           |
|                                  |             | Level 1: 1 x 10 mL/bottle; Level 2: 1 x 10 mL/bottle             | 55 mm x 28 mm x 60 mm; 41 g         | 400 mm x 270 mm x 345 mm; 6.6 kg   | 228           |
|                                  |             | Level 1: 1 x 5 mL/bottle; Level 2: 1 x 5 mL/bottle               | 55 mm x 28 mm x 60 mm; 31 g         | 400 mm x 270 mm x 345 mm; 5.5 kg   | 228           |
| Liquid Driptube Urine Control ✓X | U021-071    | Level 1: 2 x 12 mL/driptube; Level 2: 2 x 12 mL/driptube         | 130 mm x 55 mm x 55 mm; 101 g       | 385 mm x 255 mm x 320 mm; 4.7 kg   | 30            |
|                                  |             | Level 1: 1 x 12 mL/driptube; Level 2: 1 x 12 mL/driptube         | 130 mm x 55 mm x 55 mm; 62 g        | 385 mm x 255 mm x 320 mm; 3.5 kg   | 30            |
| Dry Strip Urine Control ✓X       | U021-041    | Level 1: 1 x 25 strips/canister; Level 2: 1 x 25 strips/canister | 100 mm x 51 mm x 110 mm; 126 g      | 280 mm x 280 mm x 260 mm; 3.6 kg   | 24            |
|                                  |             | Level 1: 1 x 10 strips/canister; Level 2: 1 x 10 strips/canister | 100 mm x 51 mm x 110 mm; 106 g      | 280 mm x 280 mm x 260 mm; 3.1 kg   | 24            |

✓ CE Marked for sale in the European Community 

X FDA 510(k) Cleared and CLIA Waived

**We also offer other rapid diagnostic and medical products for:**

Blood Glucose Monitoring Systems, Clinical Chemistry including Urinalysis, Immunoassay EIA/ELISA and more.

**Contact us for worldwide distribution and custom manufacturing (OEM) opportunities**

